



EXAM **V**ISION™

Quick Start

ExamVision Loupes &
Lights Accessories

EXAMVISION™



*ExamVision provides a range of products which **complement and complete your loupes.***

1. Curing filter
2. Framepart
3. Sideshield
4. Cable clip
5. Temple extender
6. Neck cord
7. Collar clip




a. Hinge
b. Mounting band

1▶

Curing Filter - Optional

To mount: Loosen screw in mounting band, until band slips into position on lamp, then tighten.

Ensure hinge is upward, as shown. Curing filter can now be flipped up and down as required.

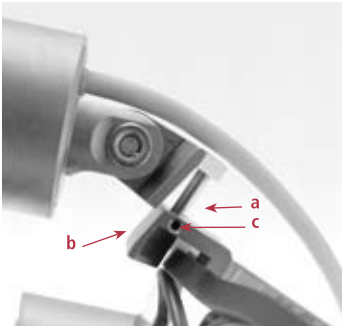


a. Post
b. Framepart
c. Connect key screw

2▶

Connect attachment Optional


With lamp pointing forwards, slide post down into hole on top of framepart. Leave open for easy removal, or tighten/lock by turning small screw in side of framepart with Connect Key provided.

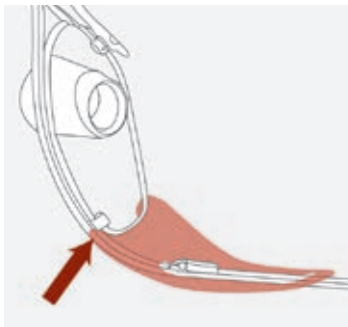

3▶

Sideshields - Optional

To attach: Bend shield and slide temple through holes. Slide into position. Tuck shield between lens and frame. Forward edge should rest just in front of loupe outer lens as shown. Fine adjust for best fit.

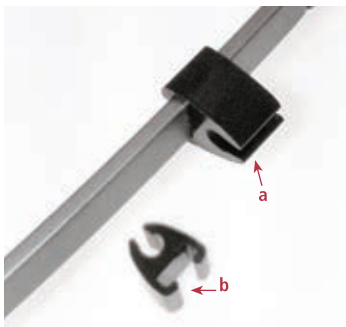


a. Cable here
b. Loupe arm here

4▶

Cable clips

To secure LED light cable: attach cable clip to loupe arm at narrowest point. Push LED cable gently into clip and smooth into place. Note: Remove carefully, to avoid damaging cable. Icon frame has integrated cable management.

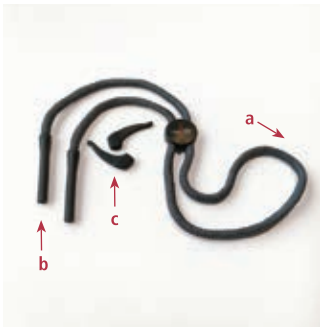

a. Neck cord
b. Silicon connector
c. Temple extenders

5 ▶ 6 ▶

Temple extenders

Optional

Temple extenders give optional extra security and tuck in behind the ears. Push onto temple tips and adjust until a comfortable position is achieved.

Neck cord

Push silicon connector over temple tips (with or without temple extenders in place). To remove, take care to grip silicon connector and pull, to avoid damaging neck cord.



7 ▶

Collar clip

Keeps the LED light cable out of the way, and can be clipped to collar or pocket, as desired.

Care and Cleaning

Your ExamVision loupes are a precision instrument. If they are damaged accidentally, or your vision through them is not perfect, please **DO NOT** try to repair or adjust them yourself. Contact your ExamVision dealer, via www.exam-vision.com for assistance.

To clean: DO NOT USE ALCOHOL! Apply ExamVision alcohol-free Custom Cleaning Solution, and polish with a soft lint-free cloth.

To disinfect: DO NOT USE ALCOHOL! Use ExamVision recommended alcohol-free Disinfectant Wipes. **DO NOT AUTOCLAVE!**



Feel free to contact your local dealer if you have any further questions
www.exam-vision.com